

A NEW ERA FOR WORKPLACE TESTING

SOTOXA™ ORAL FLUID MOBILE TEST SYSTEM



Rapid drug screening is in your hands



KEY FEATURES



PORTABLE

- · Handheld, lightweight and compact
- Carrying case stores analyzer, printer and all components
- · Optional car charging kit is available



EASY TO USE

- 3-step testing process
- Each test cartridge has a Quick Response
 (QR) code to reduce user error
- The simplicity of the device minimizes user training requirements, allowing fast implementation
- Analyzer interprets and determine the test outcome, eliminating the subjectivity of visually interpreted results



FAST RESULTS

- Collection device designed for active swabbing which speeds up collection
- Sample collection in 60 seconds or less¹
- No timed collection; volume adequacy indicator turns blue when enough sample has been collected
- Results in 5 minutes



QUALITY CONTROL

- When the analyzer is turned on, start-up checks are automatically activated
- Positive and negative quality control cartridges (PQC and NQC) check that the analyzer is interpreting results correctly
- QC testing can be enforced prior to each test, every day, week or two weeks
- On-board sensors monitor test progress
- On-board heater ensures tests run at the optimal temperature, reducing variability



DATA MANAGEMENT

- Internal storage capacity of over 10,000 tests
- Printer is provided

TEST OPTIONS

DRUG CLASS	ABBREVIATION	TARGET DRUG	CUTOFF (NG/ML)
Amphetamine	AMP	(S) Amphetamine	50
Benzodiazepines	BZO	Temazepam	20
Cannabis (THC)	THC	Δ°ΤΗС	25
Cocaine	COC	Cocaine	30
Methamphetamine	MAMP	(S) Methamphetamine	50
Opiates	OPI	Morphine	40

SOTOXATORAL FLUID MOBILE TEST SYSTEM

TESTING WHERE IT IS NEEDED

Substance abuse in the workplace can result in low productivity, accidents and injuries. An effective, simple and fast drug screening solution is key for both deterrence and identifying if drugs are being used.

INCREASE SAFETY

Keeping our work environments safe is a key priority. The SoToxa handheld technology with immediate actionable results can improve safety and help individuals and employers live their best lives.

PORTABLE, HANDHELD AND CONVENIENT


The handheld SoToxa analyzer provides the optimal combination of speed, ease of use, reliability and accuracy. The analyzer has been designed and developed to meet the high performance demands of workplace settings, which can often be in remote environments, empowering the users to take action with immediate test results.



TESTING WHERE YOU NEED IT

The SoToxa Oral Fluid Mobile Test System is the ideal instrument for drug testing in those often difficult environments. Samples can be collected anywhere without the need for gender-specific facilities. It can detect up to 6 drug classes within 5 minutes from a single oral fluid collection sample. Weighing in at under 700 g and fitting comfortably in your hand, it is a truly portable analyzer. The test system is designed to operate in a wide range of environments. The backlit, 3.2 inch full colour screen provides excellent visibility in daylight and at night.

HOW IT WORKS

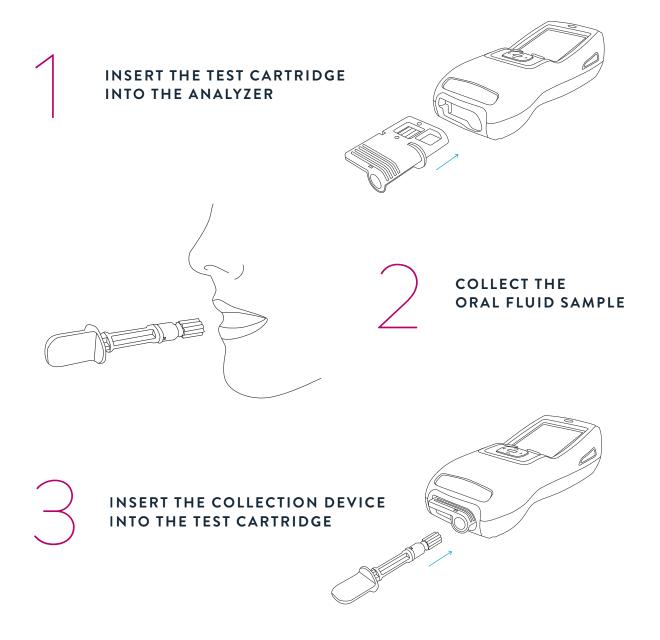



40% of Canadian drivers killed in vehicle crashes tested positive for drugs.²

SYSTEM COMPONENTS







- (1) TEST CARTRIDGE
 - Quick response (QR) code contains lot and expiry information
 - Test up to 6 drug classes in 1 convenient cartridge
- 2 PRINTER
 - Print test reports at point of test or from stored results

- (3) COLLECTION DEVICE
 - Collect samples in 60 seconds or less
 - The indicator will turn blue to indicate sufficient sample has been collected
- (4) HANDHELD ANALYZER
 - Supports multiple languages
 - LCD colour screen is readable day and night
 - Intuitive and easy to operate

Actionable test results for 6 drug classes in 5 minutes



- 1. Based on data from a non-drug using population.
- Government of Canada. Impaired driving. Updated February 4, 2021. https://www.canada.ca/en/services/policing/police/community-safety-policing/impaired-driving/drug-impaired-driving.html.



